




# Octopus Visual Field Interpretation on screen with EyeSuite Perimetry "Both eyes" display

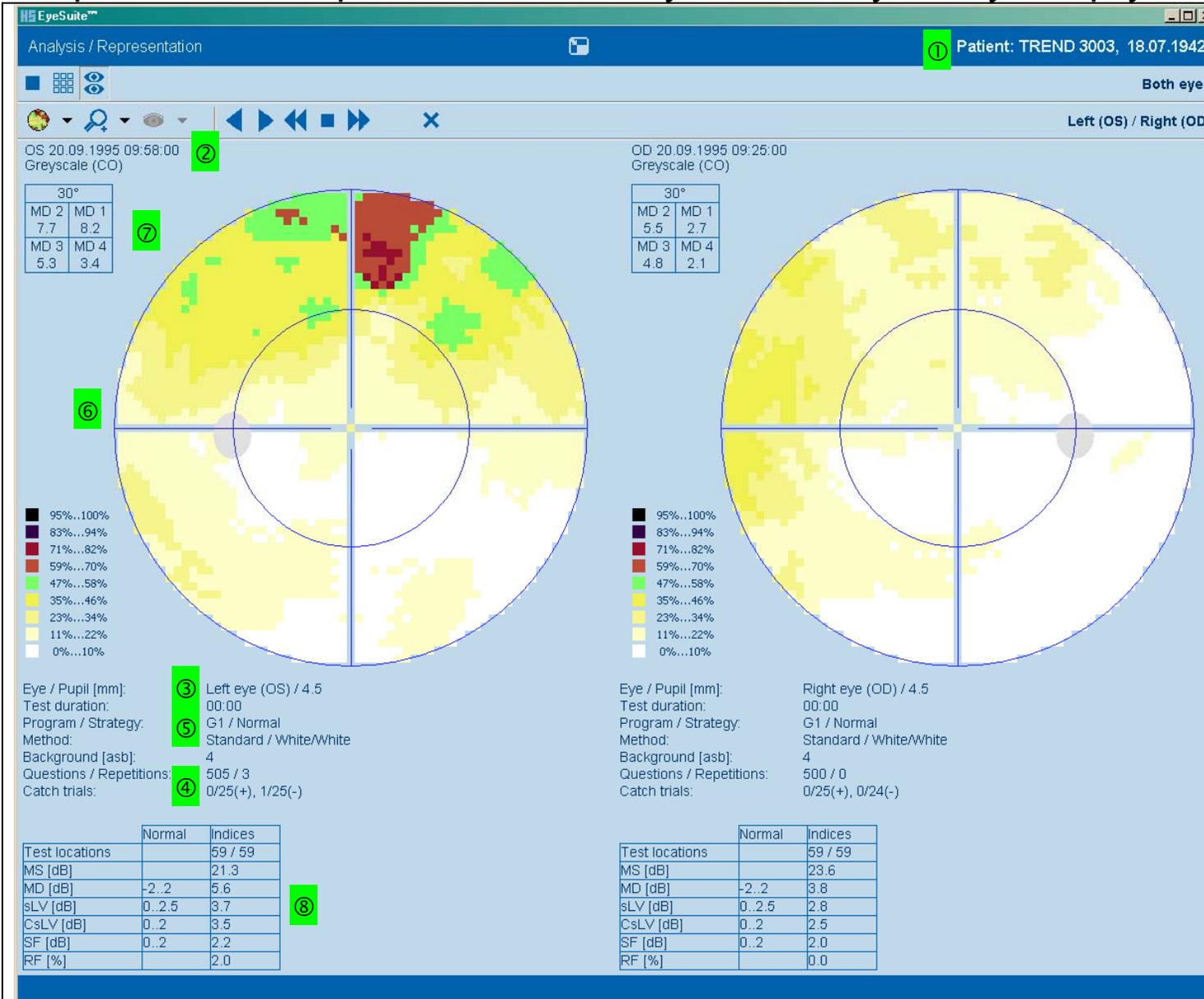

## Reliability

- ① Correct age
- ② Examination time
- ③ Pupil size  
> 3mm ok  
2.5 .. 3 mm borderline  
< 2.5mm critical
- ④ Catch trials  
< 15..20% false pos  
< % of severely de-pressed field false neg
- ⑤ Method, Strategy, Test duration

## Result

- ⑥ *Greyscale of comparisons:*  
"all white" or max. 3 small red non-cluster dots  
*Comparisons:*  
< 4 numbers with 5..8dB  
in case of cataract evaluate  
"Corrected comparison"
- ⑦ Quadrant values & left/right comparison  
 $\Delta$ MD < 1.3dB  
 $\Delta$ sLV < 0.6dB
- ⑧ Global indices  
MD < 2dB "normal" &  
sLV < 2.5 dB "normal"  
MD 2..6dB "early"  
MD 6..12dB "moderate"  
MD > 12dB "advanced"